



NEW

TODA CORONADIAG®

Rapid diagnostic test for 2019-nCoV virus (Coronavirus) in whole blood

TODA CORONADIAG® is an immunochromatographic test for the detection of the 2019-nCoV virus (Coronavirus) in whole blood samples.

PRACTICAL

- Tests provided in individual kit to optimize your consumption.
- Safe disposable lancets supplied with the tests.

RELIABLE

- High sensitivity and specificity:
 - Whole blood IgG: specificity 98% - sensitivity 100%
 - Whole blood IgM: specificity 96% - sensitivity 85%

EASY-TO-USE

- Get your results in 10 minutes from the start of the test.



EASY TO USE IN ONLY 5 STEPS

1

Open the lancet, disinfect the finger and prick it with the lancet



2

Wipe off the first drop of blood then take the next drop.



3

Pour 1 drop of blood in well S of the cassette



4

Pour 2 drops of diluent in well S of the cassette



5

Wait 10 minutes and interpret the results



CLEAR INTERPRETATION

	Positive	Positive IgG - Two colored bands appear: a band in the Control C zone and a band in the IgG zone.
	Positive	Positive IgM - Two colored bands appear: a band in the Control C zone and a band in the IgM zone.
	Negative	Positive IgG et IgM - Three colored bands appear: a band in the Control C zone, a band in the IgG zone and a band in the IgM zone.
	Invalid	Negative - Appearance of a colored band in the Control C zone and no colored band appears in the IgG or IgM zones.
	Invalid	Invalid - No band appears in Control C zone.

REFERENCES

Box of 10 individual kits: 2275-10

Vers TP07M1170220EN45
Read carefully the instructions before use. Devices not reimbursed by health insurance organizations. Document reserved for health professionals. Copyright © Toda Pharma

